


VISALYS® TEMP

1




Farbe auswählen, Kartusche einlegen, Verschluss entfernen, neuen Mischer aufsetzen.

2

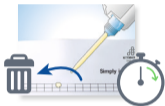

Vor jedem Gebrauch etwas Material austragen und verwerfen, Stoppuhr starten.

3

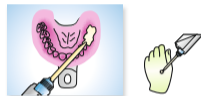

Vom Boden aus befüllen, Spitze der Mischkanüle im Material belassen.
Tipp: Erbsengroße Menge auf den Handrücken austragen (siehe Schritt 5).

4



Innerhalb von 40 Sek. nach Mischbeginn mit leichtem Druck in den Patientenmund einsetzen.

5



Aushärtegrad intraoral prüfen (oder an der Probe auf der Hand, siehe Schritt 3). Das Provisorium zwischen 1:30 Min. und 2:30 Min. nach Mischbeginn entnehmen.

6



Ausarbeitung von Visalys® Temp ab ca. 4 Min. nach Mischbeginn.




VISALYS® TEMP

1




Select shade, insert cartridge, remove cap, attach new mixing tip.

2

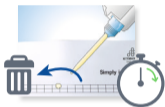

Before each use, extrude and discard a little material, start a stopwatch.

3

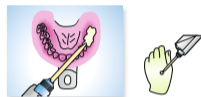

Fill from the base upwards, keep the end of the mixing tip in the material. **Tip:** apply a pea-sized quantity of material to the back of the hand (see step 5).

4



Insert into the patient's mouth, applying slight pressure, within 40 sec. of starting the mixing.

5



Check the degree of hardness intraorally (or use the sample on your hand, see step 3). Remove the temporary restoration between 1:30 min. and 2:30 min. after the start of mixing.

6



Finish the Visalys® Temp about 4 min. after the start of mixing.

